


How to use and washing instructions for
No. 96 Pill Splitter

Please issue to users

HOW TO USE

Carefully place the pill
in the bottom half of the
Pill Splitter, bring both
halves together, re-open
for split pill.

To quarter, turn the pill
and repeat the process

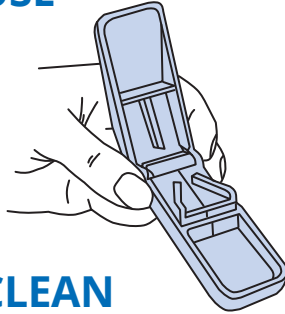


HOW TO CLEAN Sterilisation

Steam time and temperature
18 minutes at 121°C or 3-4 minutes at 134°C





Once sterilised carefully wipe the blade
as in diagram 5


To wash

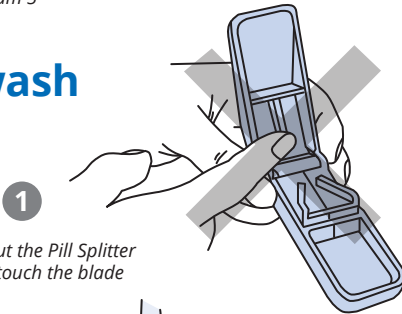

1
Open out the Pill Splitter
Do not touch the blade

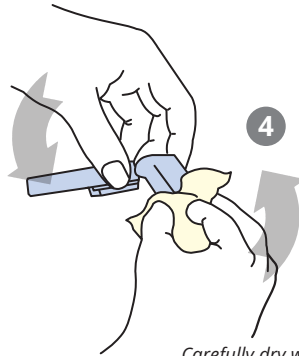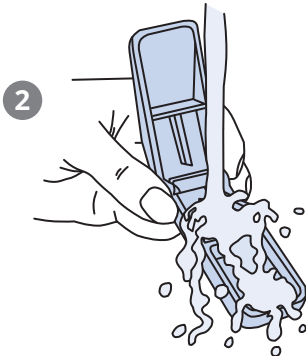

4
Carefully dry with a tissue

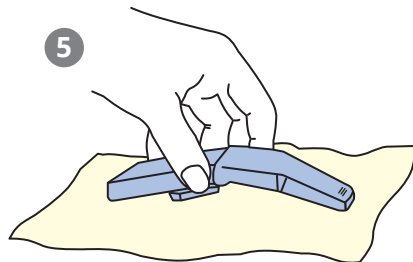

5

Caution: Contains sharp blade

This device complies with EC 1207/2001 and is Year 2000 and Euro compliant

**W+W MEDSYSTEMS
LIMITED**

2-4 Lockwood House, Lockwood Park
Brewery Drive, Huddersfield HD4 6EN

Tel: 01484 667822 Fax: 01484 667817

Email: info@wwmed.co.uk

Website: www.wwmed.co.uk